

AQUACEL® Ag Hydrofiber® Dressing with Ionic Silver



AQUACEL Ag Hydrofiber Dressing with Ionic Silver is a soft, nonwoven pad or ribbon that features the gelling benefits of Hydrofiber technology plus antimicrobial ionic silver. This dressing may be premoistened with saline for dry wounds.

AQUACEL Ag Hydrofiber Dressing with Ionic Silver is indicated for:

—Use on acute and chronic wounds, including burns, surgical wounds, diabetic foot ulcers, pressure ulcers, and leg ulcers*

Why choose AQUACEL Ag Hydrofiber Dressing with Ionic Silver?


- Ionic silver kills a broad spectrum of pathogens
- Demonstrates microbicidal activity against a wide range of pathogens
- Maintains microbicidal activity even after re-inoculation of microorganisms
- Aids in autolytic debridement (removal of nonviable tissue from the wound)
- Removed without leaving residue or causing trauma to wound bed
- Absorbs and retains exudate and harmful components
- Manages painful wounds
- Works on oncology wounds with exudate, such as fungoides cutaneous tumors and angiosarcoma

*Under the supervision of a healthcare professional.




Directions for using AQUACEL® Ag Hydrofiber® Dressing with Ionic Silver


Before dressing the wound, make sure to clean, examine, and assess the area. Debride if necessary.

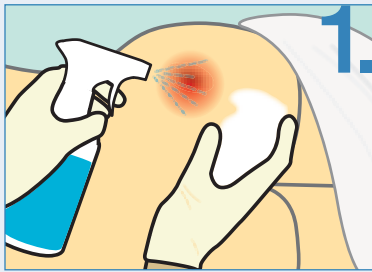

1. Use a cleansing solution on the wound surface and the surrounding skin, and dry the skin thoroughly.

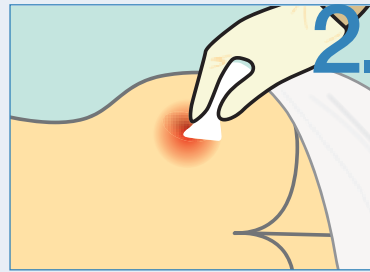

2. Loosely pack ribbon or pad into deep wounds to about 80% of the depth of the wound to accommodate swelling of the dressing.

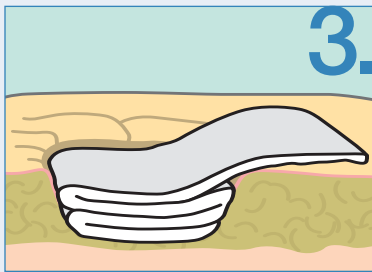

3. Leave a small overhang (at least 2.5 cm or 1 inch) to facilitate removal. Excess dressing can be cut to size without shedding fibers.



4. Cover the ribbon or pad with a moisture-retentive dressing. Secure the dressing with hypoallergenic tape, if necessary.

Dressing can remain in place for seven days maximum on chronic wounds, and 14 days maximum on partial-thickness burns

— Change the dressing when clinically indicated or when the softened area approaches the edge of the dressing

Removal of the dressing

1. Grasp one corner of the dressing and carefully lift away from wound
2. In the unlikely event of adhesion to the wound bed, the dressing can be easily removed by soaking

Ordering information

Catalog #	Size	Quantity	HCPCS
403706	2" x 2"	10 per box	A6196
403765	4" x 4.7"	10 per box	A6197
403710	6" x 6"	5 per box	A6197
403711	8" x 12"	5 per box	A6198
403712	3/4" x 18" (Ribbon)	5 per box	A6199

Examples of products that may be used with AQUACEL Ag Hydrofiber Dressing with Ionic Silver

SÂF-Clens® AF Dermal Wound Cleanser

An advanced surfactant-containing wound cleanser that washes away debris that saline may leave behind. A no-rinse formula with an adjustable nozzle.

Shur-Clens® Wound Cleanser

A wound cleanser specifically designed for use on acute, nonsurgical traumatic wounds.

DuoDERM Signal® Dressing

A thin, flexible, conformable, and moisture-resistant wound dressing that covers susceptible areas anywhere on the body to gently protect newly formed tissue or skin at risk of breakdown. Indicates with a dotted line when it's time to change the dressing.

Versiva® XC™ Dressing

A thin, flexible, conformable, and moisture-retentive wound dressing that combines the gelling action of Hydrofiber technology with foam to protect periwound skin from the risk of maceration.

Please see package insert for complete Instructions for Use.

The reimbursement information provided by ConvaTec is intended to provide general information relevant to coding and reimbursement of ConvaTec products only. Coverage and payment policies for the same insurer can vary from one region to another and may change from time to time because of ongoing changes in government and insurance industry rules and regulations. Therefore, please confirm HCPCS codes with your local DME-MAC, private insurer, or Medicaid agency before processing claims. ConvaTec does not guarantee coverage or payment of its products listed herein.

AQUACEL, DuoDERM Signal, Hydrofiber, SÂF-Clens, Shur-Clens, and Versiva are registered trademarks of E.R. Squibb & Sons, L.L.C. XC is a trademark of E.R. Squibb & Sons, L.L.C.

© 2008 E.R. Squibb & Sons, L.L.C. US-08-488 January 2008 Printed in U.S.A.

To learn more, ask your representative or call
1-800-422-8811
 M-Th, 8:30^{AM} - 8:00^{PM,ET}
 Fri, 8:30^{AM} - 6:00^{PM,ET}
 Sat, 10:00^{AM} - 2:00^{PM,ET}
www.convatec.com

 **ConvaTec**
 A Bristol-Myers Squibb Company